



### HOW TO ADD RACE DENTAL TO YOUR 3SHAPE TRIOS SCANNER



#### Thank you for choosing Race Dental for your digital scan.

Please find below instructions on how to add Race Dental to your scanner:

- 1 Please login to your 3Shape communicate account using the following link **portal.3shapecommunicate.com**
- 2 Select "Connections" from the menu at the top of the screen
- 3 Select "Add Connection"
- 4 Type trios@racedental.com.au, tick the "Show Me Labs" checkbox and click "Find"
- 5 Select "Race Dental" and click "Connect"
- 6 We will then approve the connection and you will see it become active on your connections page.
- 7 On your scanner please click the home menu button, select "Configure", "Lab Connection', select refresh.

If you require any further assistance from Race Dental, please contact our support team on **1300 722 352** or email **customersupport@racedental.com.au** 



## зshape⊳

As part of Race Dental's continuing commitment to improving digital workflows we are implementing changes to our 3Shape TRIOS Indications selections.



+ fact	+ Net					2	e Back	<b>♦</b> Net			
n Pontic	۲	Indication Crown Pontic	<b></b>				Crown Lab Preference, Any shade	$\odot$	Indication Crown	$\odot$	
vent		Material Emax	re.max cao	Emax	selmax cad	-	Abutment		Material Lab Preference	-	Lab Preference
	Te	Shade Not selected		PFM - Precious Yellow	0	5	Misc	Tes	Shade Not selected	_	Crystalite™
able	3			PFM - Precious White	9	1	Bridge	00			Opalite™
nce	V			PFM - Non-precious	0	1	Removable	4			Layered Zirconia
t Planning	8			Zirconia - monolithic	5	1	Implant Planning				e.max® CAD
nly	1			Zirconia - veneered	6		Scan only	-			e.max® Pressed
design	•			PMMA	-		Smile design	-			Lava™ Plus
					G			-			PFM-Standard-Semi Prec
											PFM-Standard-Precious
											PFM-Premium-Semi Prec
											PFM-Premium-Precious
											E.C.C. Presiour

If you require any further assistance from Race Dental, please contact our support team on **1300 722 352** or email **customersupport@racedental.com.au** 



# зshаре

As part of Race Dental's continuing commitment to improving digital workflows we are implementing changes to our 3Shape TRIOS Indications selections.





Old indications screen

#### New indications screen



If you require any further assistance from Race Dental, please contact our support team on **1300 722 352** or email **customersupport@racedental.com.au**