

HOW TO EXPORT SCANS AS STL FORMATS FROM iTERO SCANNERS

You cannot export directly from the machine and instead should use a computer




1. In your web browser, type **www.myitero.com** and log into your account.

2. Once logged in, you will see a list of your patients scanned and sent from your iTero scanner.

3. Navigate to the correct patient and click on their ID #. This will take you to the order information page.

4. Click on **"Export" (OrthoCAD 3.5 or higher)**. Now a study model will pop up and a box that says **"OrthoCAD Export"**. This will open the files and allow you to download them.

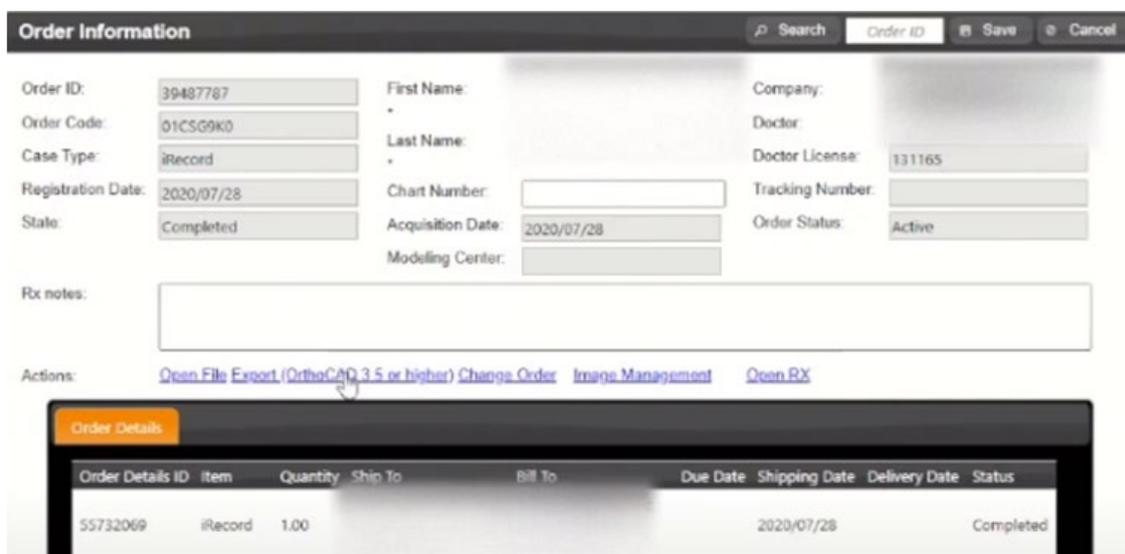


5. In the **OrthoCAD Export box**, select the setting for download. Choose the following:

Export type: Choose the “Open Shell” option.

Data Format: Choose “Two files” (arches oriented in occlusion).

Folder Name: We recommend changing this number to your patient’s name.

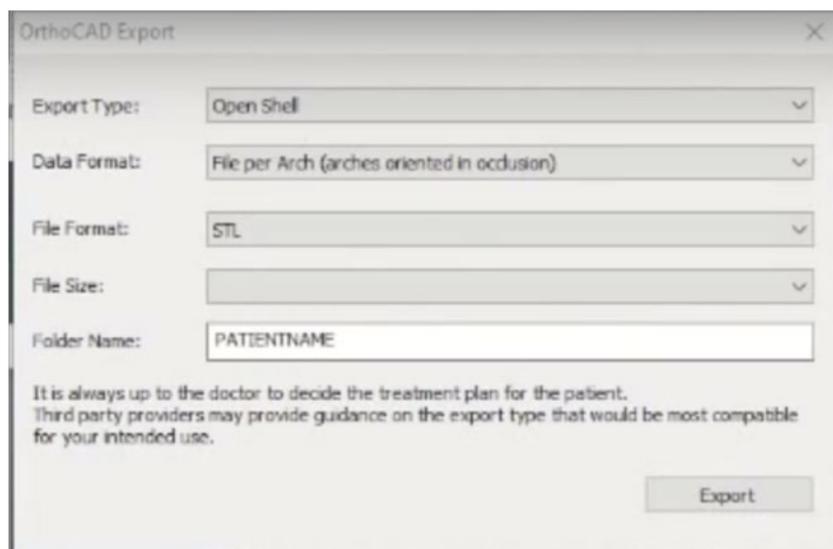

6. Click on **"Export" (OrthoCAD 3.5 or higher)**. Now a study model will pop up and a box that says **"OrthoCAD Export"**. This will open the files and allow you to download them.