



HOW TO ADD RACE ALIGNERS TO YOUR 3SHAPE SCANNER




- 1 Go to 3Shape Communicate portal <https://portal.3shapecommunicate.com/login>
- 2 Click 'Sign in' button

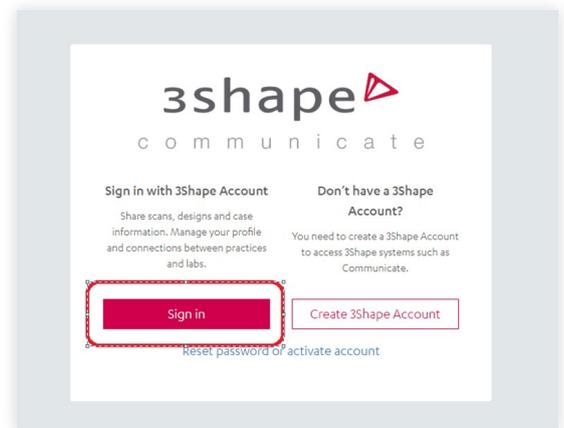

- 3 Please use your dentist's communicate account to login.

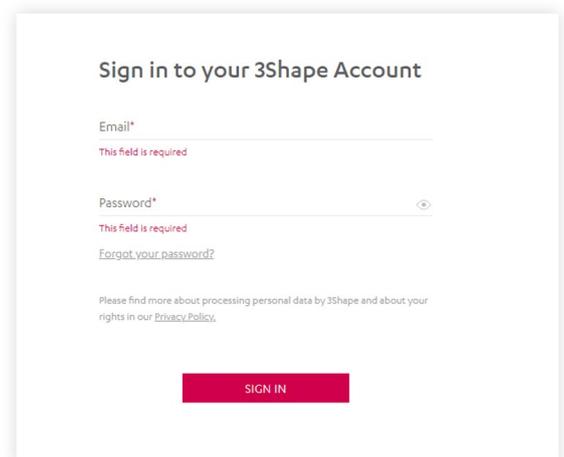


- 4 Once you logged in the 3Shape Communicate portal, please choose the tab of **'Connections'** and click **'Add Connection'** button.

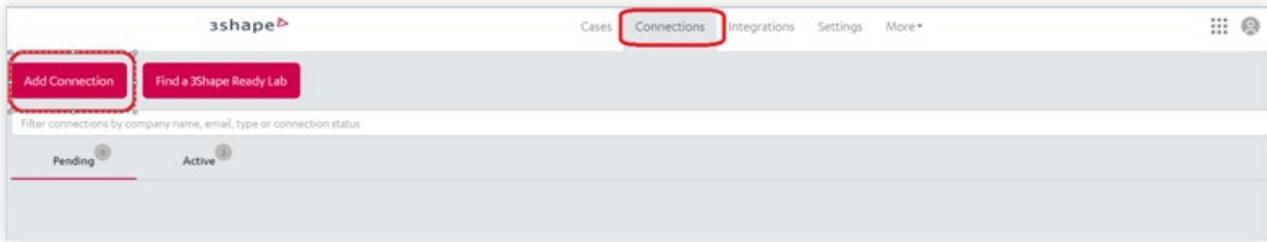

- 5 With the popped up search bar, please input Race Aligners account **'racealigners@racedental.com.au'**.




- 6 Then you will be able to see our Aligners account and please click **'Connect'** button.

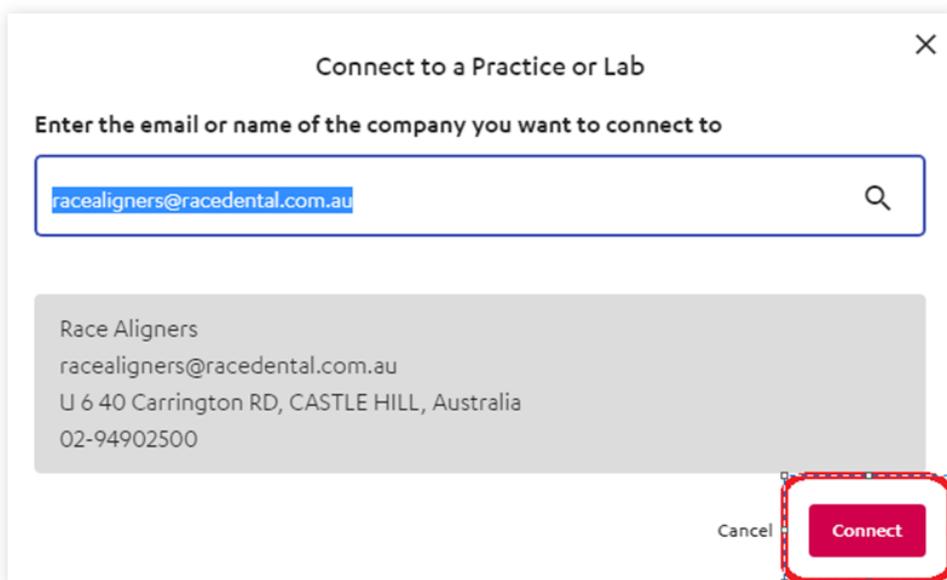


HOW TO SEND YOUR **ALIGNER** **CASE** FROM YOUR **3SHAPE SCANNER**




- 1 Set up a new patient case. Select **Race Aligners**.

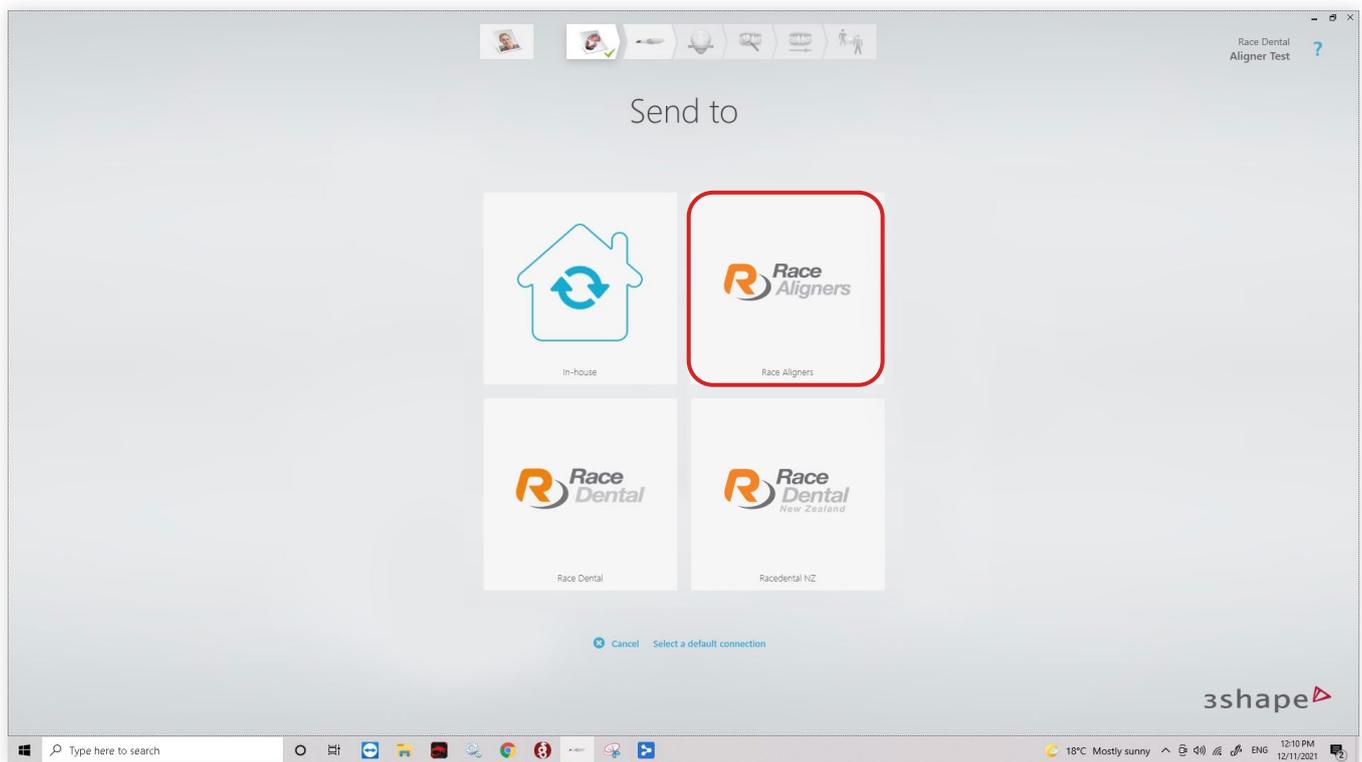

2 In the prescription select **Orthodontics**.

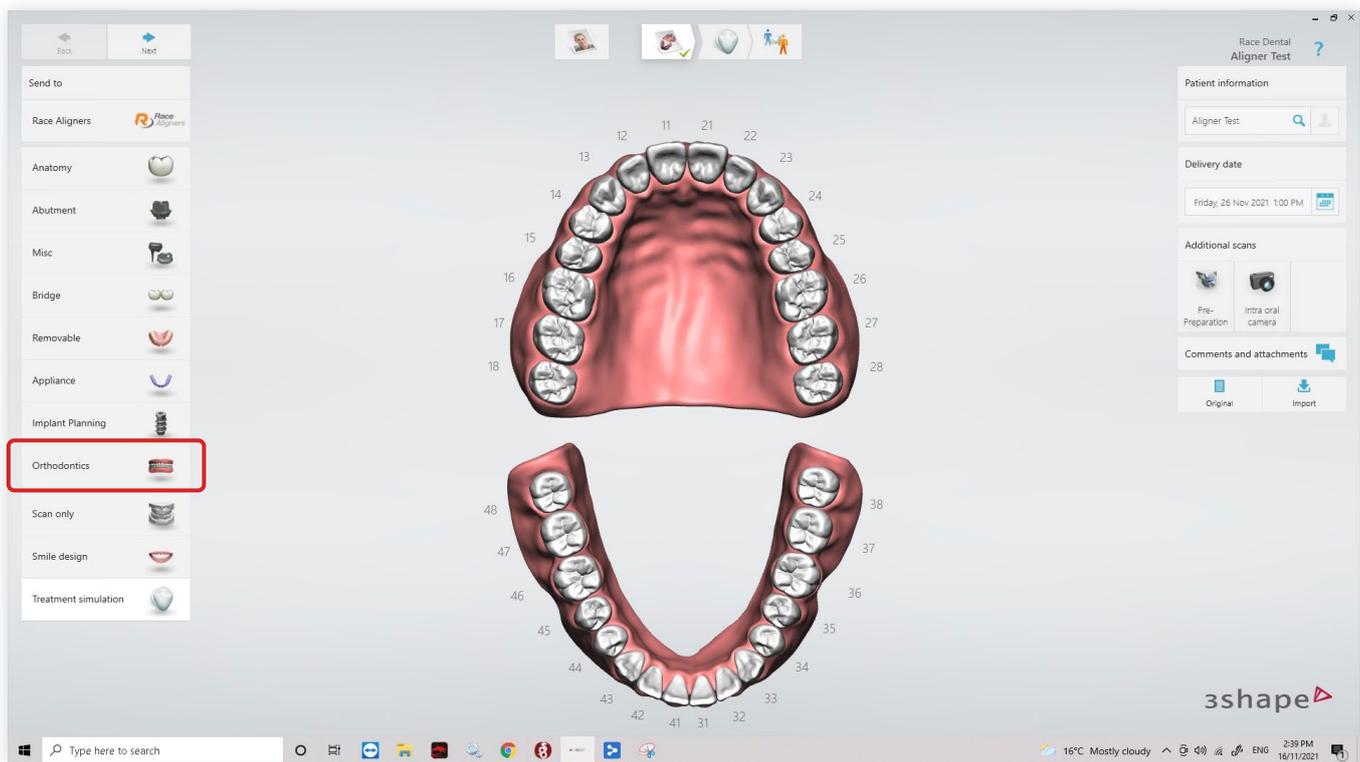


3 Then click **clear aligners**.

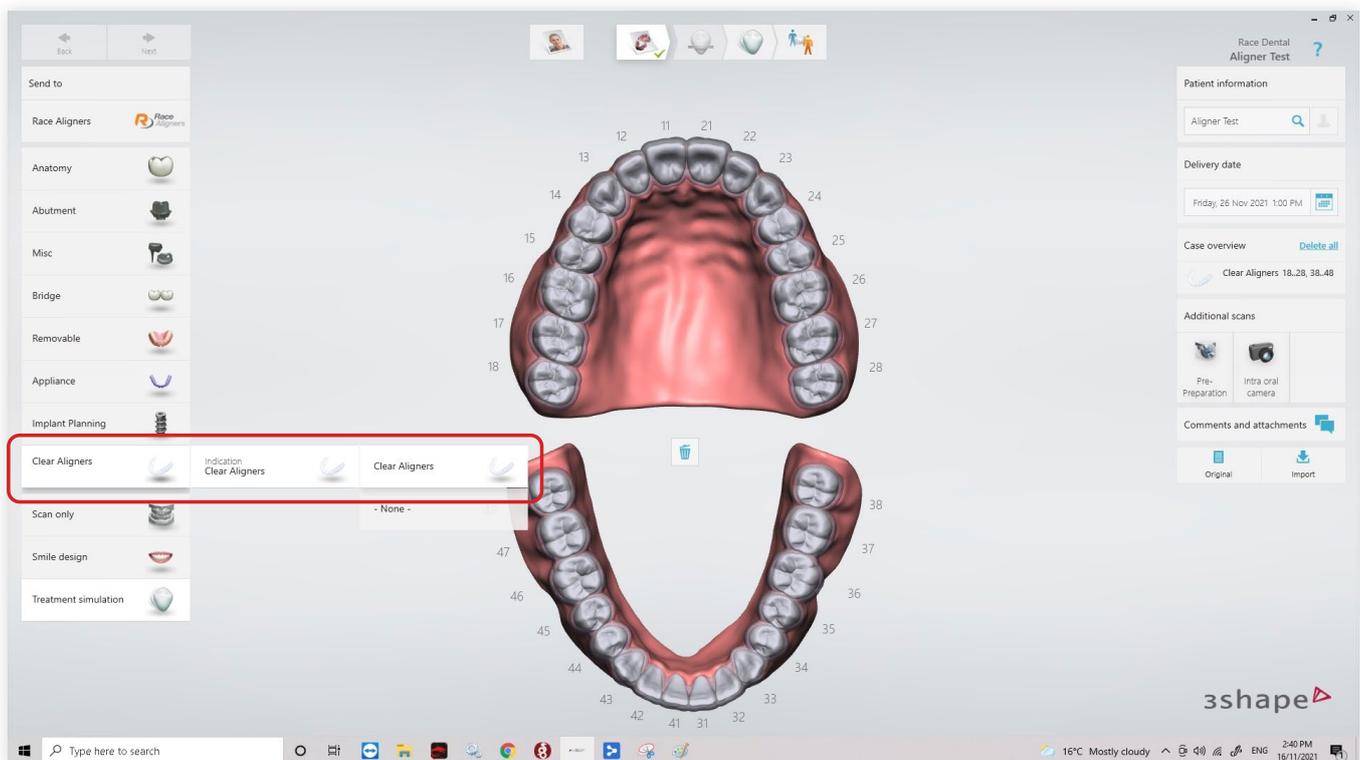

You may include any comments or attachments on the right.

4 Make sure to set a delivery date.

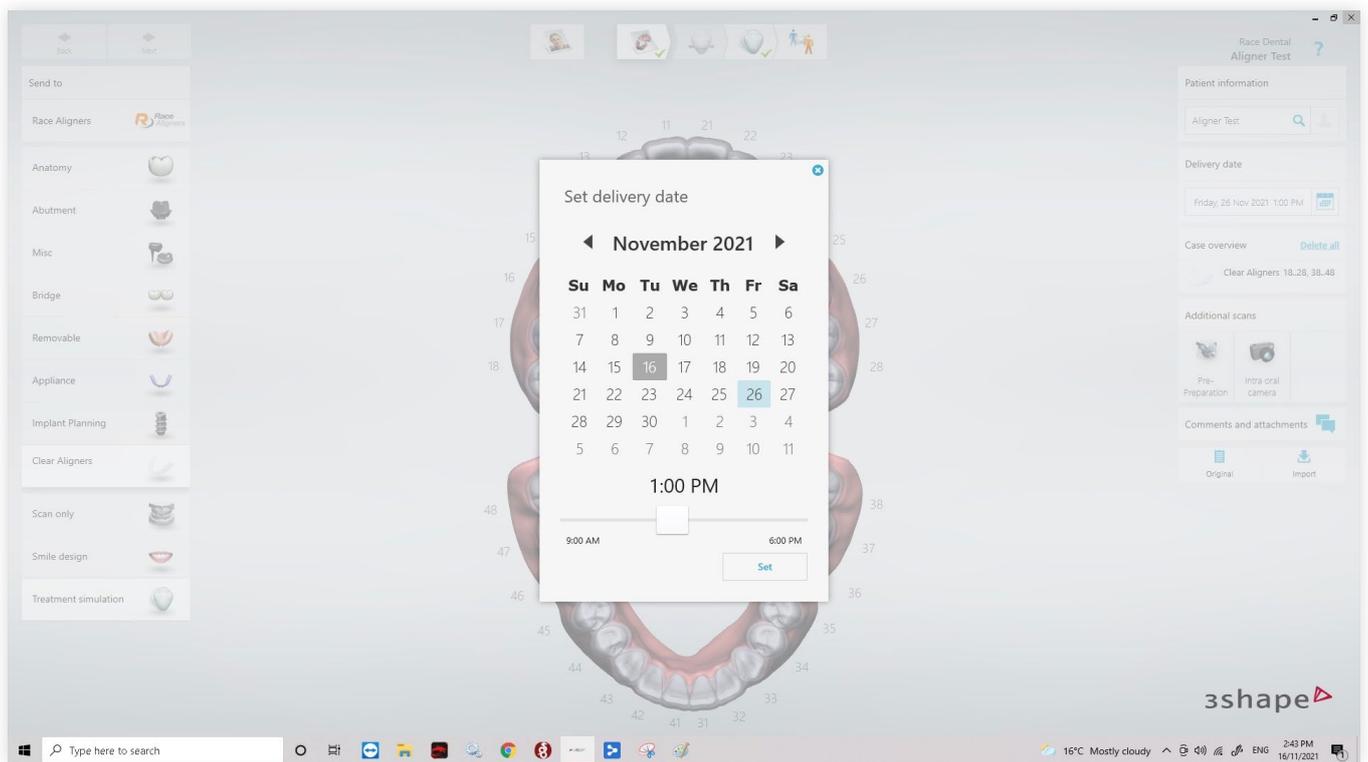

5 Once full mouth scan is completed and processed. Send Case.

