


HOW TO ADD RACE ALIGNERS TO YOUR MEDIT SCANNER




1 Go to Medit Link portal
<https://www.meditlink.com/login>

2 Please use your Medit account
to log in.

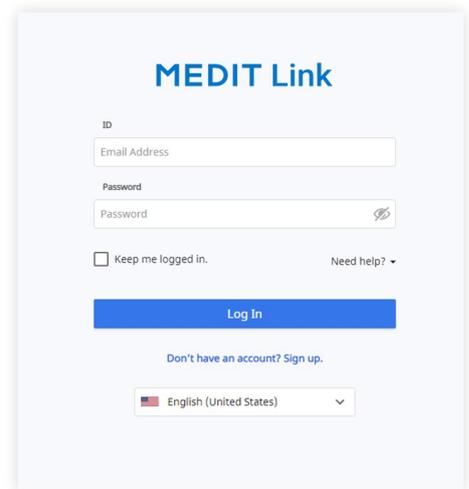


The screenshot shows the MEDIT Link login interface. It features a title 'MEDIT Link' at the top. Below the title are two input fields: 'Email Address' and 'Password'. There is a 'Keep me logged in.' checkbox and a 'Need help?' link. A blue 'Log In' button is positioned below the password field. At the bottom, there is a link for 'Don't have an account? Sign up.' and a language dropdown menu set to 'English (United States)'.

3 Once your logged in, please click
'Partners' button on the Left Sidebar.

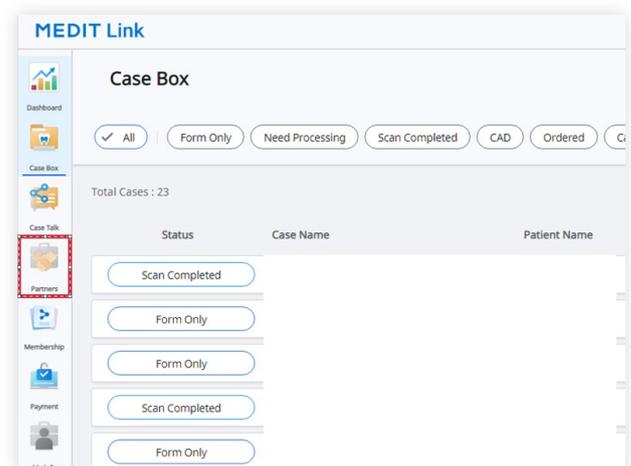


The screenshot displays the MEDIT Link dashboard. The left sidebar contains several navigation icons, with the 'Partners' icon highlighted by a red dashed box. The main content area is titled 'Case Box' and shows a filter bar with buttons for 'All', 'Form Only', 'Need Processing', 'Scan Completed', 'CAD', and 'Ordered'. Below the filter bar, it indicates 'Total Cases : 23' and shows a table with columns for 'Status', 'Case Name', and 'Patient Name'. The table contains five rows, each with a button indicating the case status: 'Scan Completed', 'Form Only', 'Form Only', 'Scan Completed', and 'Form Only'.

4 On the Partners page, please input Race Aligners account (racealigners@racedental.com.au) to search.

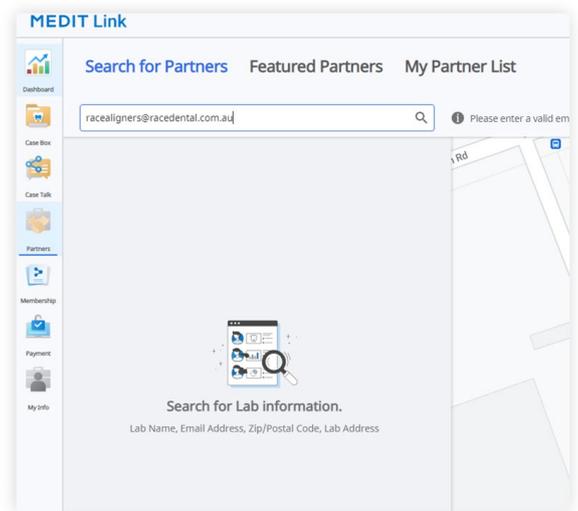


5 Then you will be able to see our Aligners account, please click **'Request Partnership'** button.

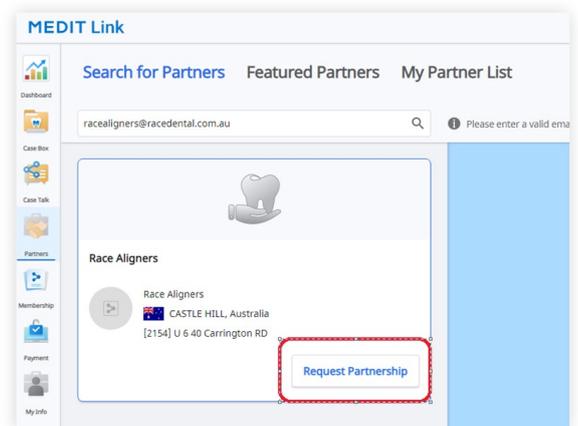


HOW TO SEND YOUR **ALIGNER** **CASE** FROM YOUR **MEDIT SCANNER**




- 1 Set up a new patient case. Make sure to select **ARCH** down the bottom.

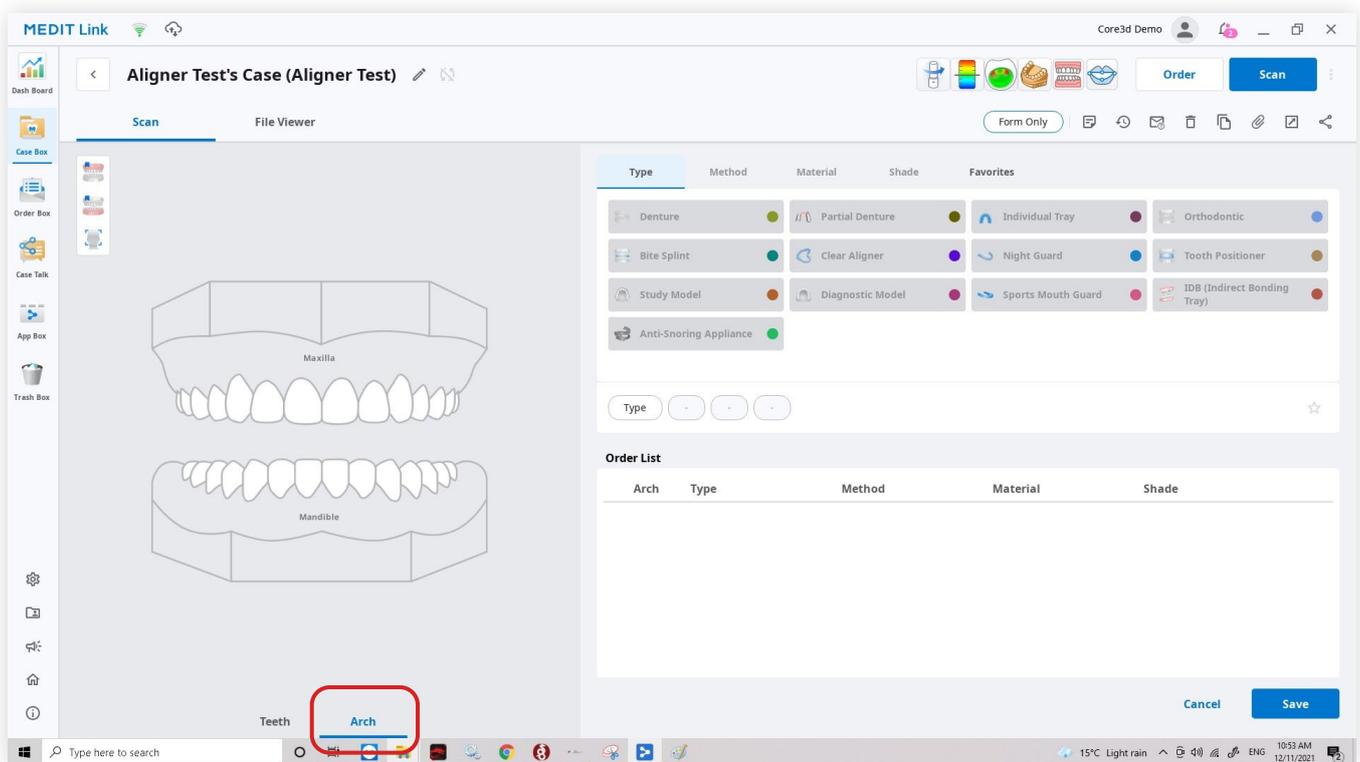

2 Select Maxilla and mandible Arch

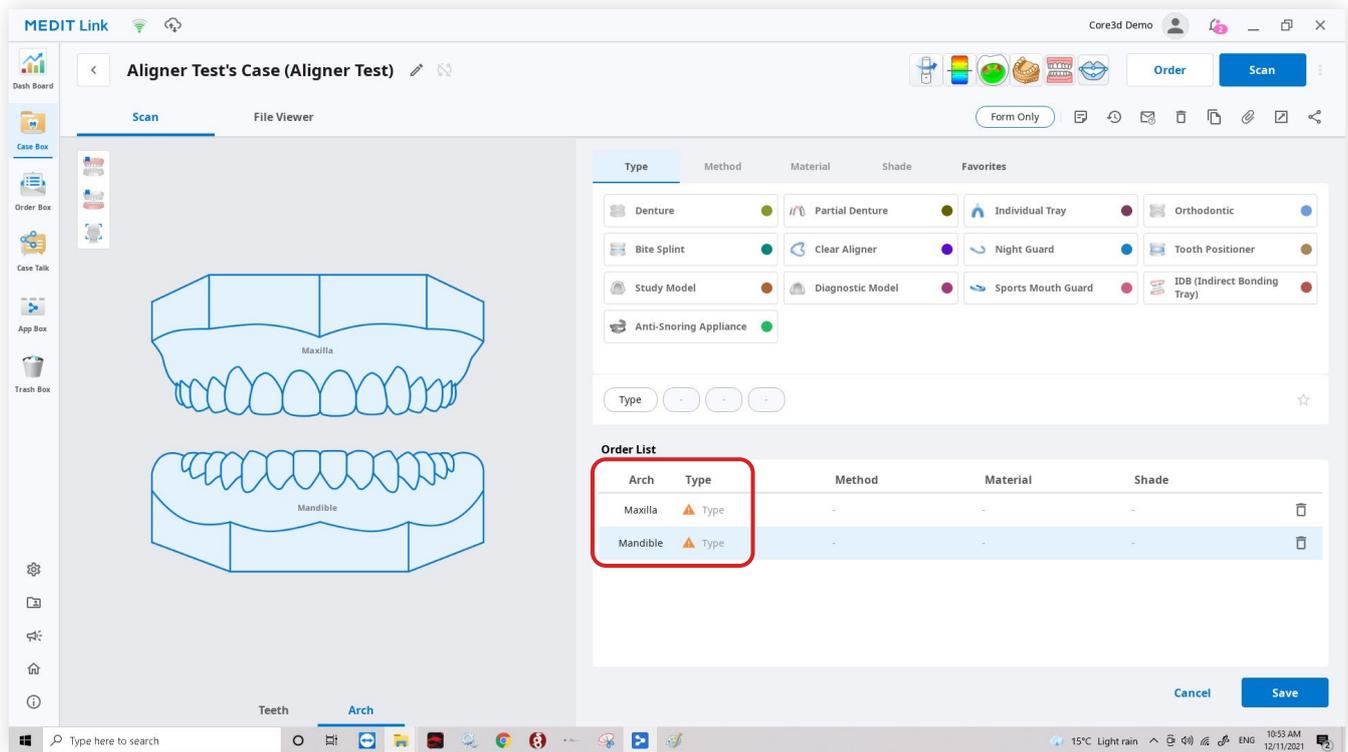

3 Under Type on the right, Select Clear Aligner and proceed with scanning .

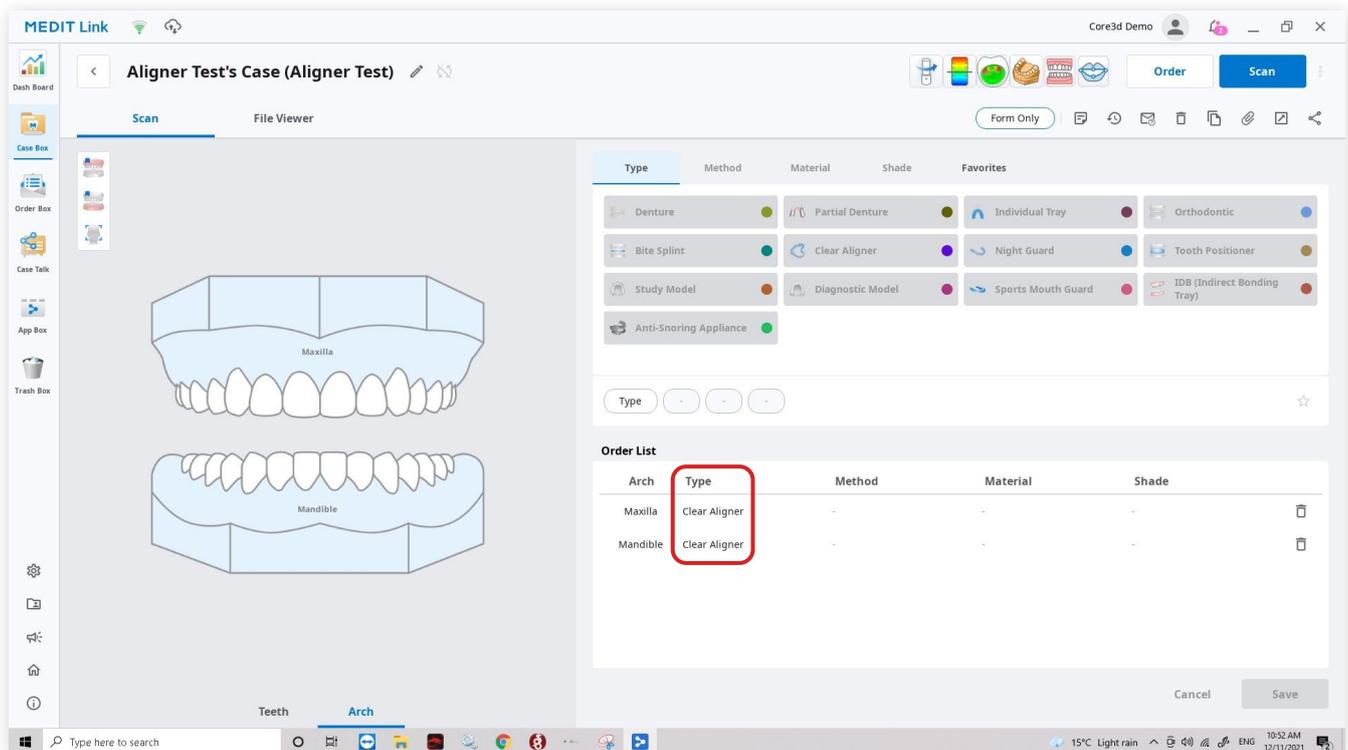

4

Once Full mouth scan is completed and processed. Select **ORDER** on top right and choose **Race Aligner partner**. Click ok to order.

